



**Alcoholic rapid surface disinfectant
READY-TO-USE**



Advantages:

- ▶ Short application times
- ▶ Extremely broad spectrum of effectiveness
- ▶ **Limited Virucidal PLUS**
- ▶ Fresh fragrances



Product type:

Ready-to-use alcoholic solution for the rapid disinfection of medical equipment as well as patient-oriented and hygienically critical areas with a wide spectrum of effectiveness and fast application times.

Type of application: As per the Medical Devices Law (MPL): Ready-for-use alcoholic quick disinfectant for the prophylactic surface disinfection of alcohol-resistant medical products for medical use. Not suitable for the final (terminal) disinfection of invasive medical products.

According to Biocides Directive (if registered): Alcoholic wipe disinfection for the disinfection of all alcohol resistant surfaces.

Indication for use:

Apply the quick disinfectant so that it covers the whole surface. Then let the disinfectant take effect, in line with the application times. Only for use on alcohol-tolerant materials (e.g. not suitable for acrylic glass). The disinfectant solution is only allowed to be used to soak compatible cloths.

Spectrum of effectiveness: Bactericidal (incl. MRSA) acc. EN 13727, EN 13697, VAH, EN 16615; yeasticidal (*Candida albicans*) acc. EN 13624, EN 13697, VAH, EN 16615; tuberculocidal (*M. terrae*) acc. EN 14348; "limited virucidal effect" (effective against enveloped viruses such as HBV, HIV, HCV) acc. DVV; "**limited virucidal PLUS**" (enveloped viruses and Adeno-, Rota- and Noro-viruses) acc. DVV and EN 14476.

Application times

(related to the impregnating liquid):

Bacteria as per EN 13727 and EN 13697 (high contamination):	conc. 1 min.
<i>Candida albicans</i> as per EN 13624 and EN 13697 (high contamination):	conc. 1 min.
TbB (tuberculosis) inactivation (<i>M. terrae</i>) as per EN 14348 (high contamination):	conc. 1 min.
Enveloped viruses (e.g. HBV, HCV and HIV) as per DVV/2012 in an area test with high contamination:	conc. 1 min.
Limited virucidal PLUS (high contamination):	conc. 1 min.
Adeno-, Rota- and Noro-viruses (EN 14476, high contamination):	conc. 1 min.



Application times

(related to the impregnated wipes):

Bacteria and candida albicans (as per VAH, version: 02.04.15 and EN 16615, high contamination)*: conc. 2 min. *Wipe-Impregnation-System listed for 5 min. in the VAH-list.

Ingredients (100 g contains):

43.0g ethanol, 8.0g 1-propanol, 0.055g didecyl-dimethylammonium chloride.

Chemical-physical data:

- ▶ Fragrance: lemon, fruit, flower, neutral
- ▶ Density at 20°: 0,912 g / cm³
- ▶ pH value at 20°: <10,0
- ▶ Appearance: colorless, clear liquid

Safety notes:

H 226 Flammable liquid and vapour. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Wear eye protection/face protection. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Store in a well ventilated place. Keep container tightly sealed. Dispose of the contents/container in accordance with local/regional/national/international regulations. Flammable! Do not spray into naked flames! Flash point as per DIN 51755: approx. 25 degrees. Application max. 50ml per m2. The total applied amount per room must not exceed more than 100ml per m2 of room area. Keep the containers tightly sealed. Avoid eye contact.

Notes on disposal:

Only put the packaging into the recycling system if it is empty of residue. Dispose of residual product in accordance with EAK 07 06 99. Directive 648/2004/EC: Contains < 5% cationic surfactants, fragrances.

National / International Test Standards:

Dentalrapid SD liquid neutral, lemon, flower, fruit fulfill the following national and international test standards: EN 13727, EN 13697, VAH, EN 16615, EN 13624, EN 14348, EN 14476; enveloped viruses (RKI / DVV).

Delivery forms



Packing Unit	Neutral REF	Lemon REF	Fruit REF	Flower REF
100 ml spray bottle	71000100	71010100	71020100	71030100
500 ml bottle	71000500	71010500	71020500	71030500
1 L bottle	71001000	71011000	71021000	71031000
5 L bottle	71005000	71015000	71025000	71035000

Accessories:

Product Type	REF
1 litre metered-dose squeeze bottle	701000904
Spray nozzle white for 500 ml and 1000 ml bottle	701000903
Dosingpump for 5 L + 10 L canister	701000912
Canistercock for 5 L + 10 L canister	701000911
Canister wrench	701000909

Shelf life: 3½ Years from the date of manufacture.

Contact:

Further information about reports / test results and about the 360° hygiene program is available on request.

Tel.: +49 (0) 2266 4742 - 0

E-Mail: info@mueller-omicron.de



Made in Germany by

Müller-Omicron GmbH & Co. KG

Schlosserstr. 1

D-51789 Lindlar

Tel.: +49 (0) 2266 4742-0

Fax: +49 (0) 2266 4742-23

www.mueller-omicron.com

